# Vector® Fluid polish

DE	Gebrauchsanweisung	SL	Navodila za uporabo
EN	Instructions for use	HR	Upute za upotrebu
FR	Mode d'emploi	HU	Használati útmutató
IT	Uso	EE	Kasutusjuhend
ES	Instrucciones para el uso	LV	Lietošanas pamācība
PT	Instruções de uso	LT	Naudojimo instrukcija
NL	Gebruiksaanwijzing	RU	Инструкция по применению
SV	Bruksanvisning	BG	Упътване за употреба
FI	Käyttöohje	RO	Instrucțiuni de utilizare
NO	Bruksanvisning	TR	Kullanma talimatı
DA	Bruksanvisning	UA	інструкція із застосування
EL	Οδηγίες χρήσης	KZ	қолдану жөніндегі нұсқау
PL	Instrukcja użytkowania	UZ	Ko'rsatmalar
CS	Návod k použití	AZ	İstifadə təlimatı
SK	Návod na použitie	AM	օգտագործման ուղեցույց՝

 orochemie GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim, DE  
a DÜRR DENTAL company  
Tel. +49 7142 705-0  
info@duerrdental.com  
www.duerrdental.com  
Made in Germany

 **DÜRR  
DENTAL**

DE

# Vector® Fluid polish

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# Vector® Fluid polish

## Instructions for use

**Vector® Fluid polish** is a medical device. This product is only intended for professional dental applications together with the Vector® System of Dürr Dental SE and must only be used by trained specialists. The formulation is free of animal ingredients, gluten, lactose, fructose and microplastics.

**Indication:** Vector® Fluid polish is a suspension for use in the Vector® System. that gently and intensively cleans tooth, denture, and root surfaces with ultrasound dynamics. Due to this minimally invasive treatment with ultrasound dynamics, gingival pockets in particular are thoroughly flushed and the non-adhering plaque they contain is removed. Biofilms and subgingival calculus are also gently removed. Bacteria and endotoxins are reduced. Vector® Fluid polish is suitable for children and adults. Do not swallow.

**Contre-indication:** Vector® Fluid polish is contraindicated in patients with known allergies to ingredients of Vector® Fluid polish (e.g. limonene).

**Composition:** Vector® Fluid polish contains hydroxylapatite, calcium phosphate, dispersing agents, PHB ester, adjuvants and aroma (contains limonene).

**Application:** Vector® Fluid polish is a concentrate for sole use with the Vector® System of Dürr Dental SE, that is automatically diluted with water to the correct usage concentration.

Follow the instructions for use of the Vector® System precisely. Shake before use.

**Warning:** Do not swallow. Avoid contact with the eyes. If there is contact with the eyes, rinse thoroughly with water. Do not remove the cap from the pouch. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

**Precautions:** The patient and operator need to wear suitable eye protection. The operator additionally needs to wear mouth and nose protection and suitable protective gloves.

**Storage and shelf life:** Store in enclosed, well-ventilated rooms away from moisture and bright light between 5°C and 25°C. When the storage conditions are observed, the shelf life is 18 months in the sealed container and 4 weeks after opening.

**Date of opening:** enter on the back of the pouch (white area).

**Disposal:** Dispose of in accordance with the local official regulations

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